



**B&B DENTAL**  
implant company



## PROSTHETIC PROCEDURES

### BASIC INFO

DURAVIT IMPLANT LINES  
3P - EVOLUTION - WIDE

# TRY-INN KIT ABUTMENTS

Try-INN kit abutments helps the dental technician to select the most suitable abutment, based on the inclination and the transmucosal height of the implant that has been inserted.




Try-INN abutments are color-coded, well-marked on the holder and easily readable.

The box contains 3 pcs. of each code (see table beside) for a total of 27 abutments.



0° Ref.      15° Ref.      25° Ref.

H1	PC107.01/1	PC107.02/1	PC107.03/1
H2	PC107.01/2	PC107.02/2	PC107.03/2
H3	PC107.01/3	PC107.02/3	PC107.03/3

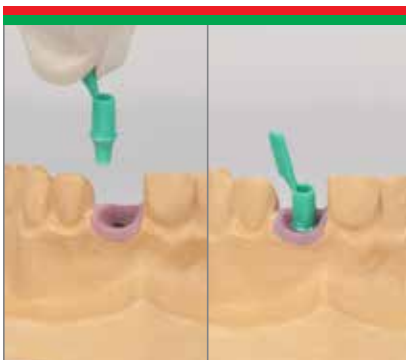

Place the try-inn abutment on the implant (intra-oral use) or on the implant analog (extra-oral use).

This will aid in checking the gingival height (H1, H.2 e H.3 mm) and axial alignment of the potential restoration (0°. 15° e 25°).



TRY-INN abutments are fabricated in sterilizable polymer material. Easy to handle thanks to the plastic holder.



Turn the plastic kit upside down to read the corresponding ø5 titanium abutment.